Calligator

Special rinse aid for use in DISCHER bedpan washer-disinfectors Liquid concentrate



Main field of application: Against lime precipitation in steam generators of bedpan washer-disinfectors

Properties: Calligator is a mildly acidic rinse aid with special components for effectively

complexing water hardness. Due to the good lime bonding ability and the excellent wetting properties the items to be washed are rinsed stain-free and

without lime deposits.

When used as intended no lime deposits develop in steam generators or in

pipes.

Calligator can be used with all degrees of water hardness.

Application and dosage: Calligator is used in DISCHER bedpan washer-disinfectors.

> Dosage: 0.5 – 1.5 ml/l, depending on water hardness via integrated dosing devices. Dosing systems and steam generators must be acid-compatible. This has been tested and approved for DISCHER bedpan washer-

disinfectors.

Do not mix with other products.

Rinse out dosing system including suction hose with water before changing

product.

Reprocessing should comply with all ordinances pursuant to the regulations on medical devices and should be performed with appropriate validated

processes.

Please observe the reprocessing information of the medical device manufacturers according to the requirements of the DIN EN ISO 17664.

The instructions given by the manufacturer of the washer-disinfector are to

be observed.

For professional use only.

Technical data: Density (20 °C): approx. 1.1 g/cm³

pH-range (determined in deionised water, 20 °C) 0.5 – 1.5 ml/l: 3.4 – 3.1

Viscosity (concentrate, 20 °C): < 50 mPas

Ingredients: Ingredients according to Regulation (EC) No 648/2004 on detergents:

< 5 % polycarboxylates, also: preservatives (methylchloroisothiozolinone

/methylisothiazolinone)

CE-mark: (EMD Calligator complies with European guidelines for medical devices.

If a serious incident occurs with the product, report it to the manufacturer and

the relevant national authority.

Storage information: Store in a frost-free place. Always store at a temperature between 0 °C and

> 30 °C. Usable for 3 years when stored as recommended. For expiry date refer to the stamp mark on the label behind the hourglass symbol \square .

For safety information see Safety Data Sheets.

Hazard and precautionary statements:

> Dispose only when container is empty and closed. For disposal of product residues, refer to the Safety Data Sheet.

DISCHER Technik GmbH • Fuhr 4-6 • 42781 Haan - Gruiten • Germany

Calligator

Special rinse aid for use in DISCHER bedpan washer-disinfectors Liquid concentrate



Distribution: Distributed by: DISCHER Technik GmbH, Fuhr 4-6, 42781 Haan-Gruiten,

Germany, Tel: +49 (0) 2104 2336-0, info@discher.de, www.discher.de

Manufacturing: Manufactured by: Chemische Fabrik Dr. Weigert GmbH & Co. KG,

Mühlenhagen 85, 20539 Hamburg, Germany, Tel.: +49 (0)40 7 89 60-0

With the above information, which is appropriate to the current knowledge we describe our product regarding possible safety necessities, but we do not involve any quality description or promise certain properties.

Date of issue: 11/2021